LIFETIME LIMITED WARRANTY

Your Guardian product is warranted to be free of defects in materials and workmanship. This warranty is limited to the original consumer purchaser.

This warranty does not cover product failure due to owner misuse or negligence or normal wear and tear. The warranty does not extend to non-durable components, such as but not limited to rubber accessories and grips which are subject to normal wear and tear and replacement. This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

In the event of a defect covered by this warranty, we will at our option repair or replace the product. For warranty service please contact the retailer where the product was purchased. If you have a question about your Guardian brand product or this warranty, please call us toll-free at 800.333.4000.

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GUARDIAN®

Walkers

Children's Folding Models 7749, 7750
Tyke Non-Folding Models 7751, 7752
Strider Models 7780, 7781, 7782, 7783

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Garantía

IMPORTANT SAFEGUARDS

Important Information is highlighted by these terms:

WARNING– Important safety information for hazards that might cause serious injury.

CAUTION– Information for preventing damage to the product.

NOTE– Information to which you should pay special attention.

READ ALL INSTRUCTIONS BEFORE USING.

⚠ WARNING

For safety, these precautions must be taken when using this product. Before each use make sure:

1. Walker is adjusted to correct height.
2. Locking mechanisms and spring buttons are fully engaged.
3. Tips are in good condition. Replace tips immediately when worn or missing.
4. Make sure the walker sits level and all legs are adjusted to an equal height.
5. Use only as instructed by your healthcare provider.

⚠ WARNING

Always use extreme care when using this product. Avoid all potential safety hazards including slippery, uneven or soft surfaces and objects in path.

⚠ WARNING

Use wheels on front legs of walker only with the exception of Rear Wheel Brakes.

⚠ WARNING

Do not "rack" the walker by pushing one side forward and then the other side without lifting the walker.

⚠ WARNING

Use Guardian accessories only. Use of other accessories may result in improper fit and risk of fall and injury.

⚠ WARNING

Do not use on stairs unless otherwise instructed by a professional.

⚠ WARNING

The use of walkers in water (showers, bathtubs, pools, etc.) may corrode the working mechanisms of your walker. The use of a corroded walker may result in damage to the walker and/or injuries to the user.

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⚠ WARNING

Consult your healthcare professional to select the right walker and accessories for you and for specific use instructions based on your abilities and daily activities.

⚠ WARNING

When transferring from a wheelchair to a walker, be certain wheelchair wheels are locked.

⚠ WARNING

Do not exceed the maximum weight capacity listed on your product.

FAILURE TO FOLLOW THESE INSTRUCTIONS MAY RESULT IN SERIOUS INJURY.

SAVE THESE INSTRUCTIONS.

HEIGHT ADJUSTMENT

1. Adjust the walker so that when the arm is extended, the handle is at wrist height.
2. Depress spring button and slide leg up or down.
3. Engage spring button into hole that provides correct height.
4. Repeat for each leg, making sure walker is level.

CLEANING

Grip Cleaning Instructions

Clean the grips with a mild soap, detergent or household cleaner. Do not use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause the grip to loosen.

NOTE– Tips on this product contain dry natural rubber.

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© 2002, Sunrise Medical 12.02