



Quickie TS in Tilt & Recline

QUICKIE®

Quickie® TS



features & benefits

- With the addition of the “new” short frame option and the growing frame option, a wide range of adjustable sizes are now available.
- The flared front end increases stability while providing additional clearance for the “new” Contracture Footrest System.
- To meet any positioning need. Along with 45° of tilt, there is an option for up to 60° of recline.
- The tilt limiting function lets you control the tilt range from 45° to 15° in 6° increments.

 **SUNRISE
MEDICAL**

www.sunrisemedical.com

Quickie

Quickie TS



Quickie TS and the New Contracture Footrest System Option**

- Contracture Footrest seat frame (hanger tubes removed)
- Flared front end base frame
- Width-, depth- and angle-adjustable hanger tubes
- Multi-directional angle-adjustable aluminum footrest plates

**See Quickie TS Order Form or Contracture Footrest Flyer/Order Form.




Seat Pan

This seat pan option allows the Quickie TS or Quickie T45 to be compatible with a variety of cushions.




Dual-Post Height-Adjustable, Flip-Back Armrests

These flip-back padded armrests can be moved up or down with ease. All dual-post armrests are available with 10" desk- or 14" full-length pads.

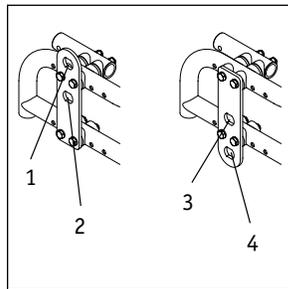

Four-Position Axle Plate

See Rear Seat Height chart in Specifications

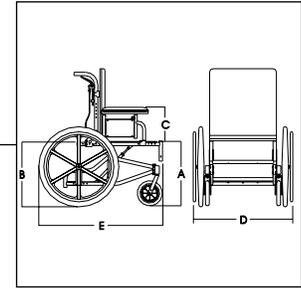


(Seat moved forward to foremost position and the 24" rear wheels moved to rearward most position)

Quickie TS


Specifications

Weight: 49 lbs.
Weight Capacity: 250 lbs.

Tilt Angle	D
15	1
21	2
27	3
33	4
39	5
45	6

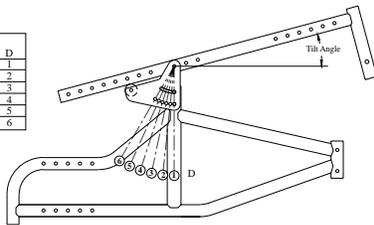

Tilt Limiting Adjustment Range

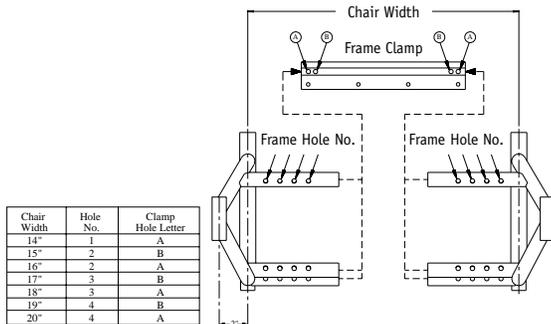

Chair Width	Hole No.	Clamp Hole Letter
14"	1	A
15"	2	B
16"	2	A
17"	3	B
18"	3	A
19"	4	B
20"	4	A

Chair Width Adjustment Range

ACTUAL MEASUREMENTS MAY VARY ACCORDING TO CHAIR CONFIGURATION AND ACCESSORIES. PLEASE CONSULT ORDER FORM FOR COMBINATIONS AND RESTRICTIONS. SPECIFICATIONS SUBJECT TO CHANGE WITHOUT NOTICE.

Front Seat Height (A) (with seat pan)

Caster Fork Mounting Hole	5" LP Poly 5 1/4" Frk	5" LP Poly 6" Frk	5" LP Poly 7" Frk	6" Poly/Pneu 5 1/4" Frk	6" Poly/Pneu 6" Frk	6" Poly/Pneu 7" Frk	8" Poly/Pneu 5 1/4" Frk	8" Poly/Pneu 6" Frk	8" Poly/Pneu 7" Frk
Top	16"	16.75"	17.75"	16.25"	17"	18"	17.25"	18"	19"
Bottom	17"	17.75"	18.75"	17.25"	18"	19"	18.25"	19"	20"

Rear Seat Height (B) (with standard four-position axle plate)

Rear Hole Position	12" Pneu	16" Pneu	20" Pneu	22" FP Poly	22" Pneu	24" LP Poly	24" FP Poly	24" HPr Clnchr	24" Pneu & Kevlar
Position 1	N/A	15"	17.5"	18.75"	18.75"	19.25"		18.75"	
Position 2	15.5"	17.5"	20"	21"	21"	18.25"		21"	
Position 3	17.25"	19"	21.5"	22.5"	22.5"	19.25"		2.5"	
Position 4	19.5"	21.25"	23.75"	25"	25"	18.25"		25"	

Armrest Height (C)

Fixed Flip-Back	Ht.-Adj. Flip-Back	Dual-Post Ht.-Adj.	Adj. Lckng Flip-up	Footrest Height (Not Noted in Diagram)		
				Swing-Away	ELR/ALR	Heavy-Duty/Lift-Off
9.5"	10.25"-15.25"	9.5"-14.5"	6.5"-13"	14.75"-17.25"	15.75"-18.25"	15"-18.25"

*Measured From seat pan to top of armrest pad. Without cushion.

Overall Width (D) (With dual-post armrests)

Frame Width	14"	15"	16"	17"	18"	19"	20"
Mag Wheel With Aluminum Handrims	22.5"	23.5"	24.5"	25.5"	26.5"	27.5"	28.5"
Spoke Wheel with Aluminum Handrims	23.5"	24.5"	25.5"	26.5"	27.5"	28.5"	29.5"

Overall Length (E) (With dual-post armrests)

Frame Depth/Back Post Location	Overall Length
Short: 14"-17"	36"
Reg: 16"-19"	38"
Long: 19"-22"	41"