

power mobility



GUARDIAN®





Microlite Ruby 11

Features of the Microlite

- **Disassembles easily** into three lightweight components. By using a lightweight tubular design, easily detachable seat and quick-release battery, the Guardian Microlite Ruby and Ruby 11 quickly break down into three manageable components. Ideal for travel.
- **Quick-Release Battery** eliminates the frustration of breaking down the chair to access the batteries and undoing stubborn battery cables.
- **Built-In Travel Handle**, like a suitcase on wheels, allows the Microlite base frames to be easily moved across a level surface.
- **Easy-to-reach Freewheel Release** levers point inward to prevent accidental release. Only a minimum amount of force is required to engage the levers.
- **Five Different Speed Settings** are available. These options give the user the ability to control the chair's performance in a variety of environments.
- **Convenient Charging** through the joystick or the removable battery, allowing the user to leave everything but the battery in a storage location when charging.
- **Small Base Frame** (or footprint), is ideal for use in confined living spaces.
- **Flip-down/Flip-up Armrests** aid transferring in and out of the chair and eliminate the risk of misplacing the armrests.
- **Battery Box Features** a circuit breaker, on/off switch, battery level indicator, charging port, flip-down handle and set of outdoor reflectors. These combined features provide a high level of safety and convenience.
- **Flat-free Tires!**



Microlite Ruby



Easy disassembly for transport and storage

The Guardian Microlite family is lightweight, highly portable and allows seniors to experience the freedom of power mobility at an affordable price. The Guardian Microlite is the right product for anyone wanting an ultra-maneuverable, ultra-transportable mobility solution.



Features of the Aspire

- **Top speed of 6.0 mph** (full) and 5.5 mph (mini) puts the Guardian Aspire among the fastest center wheel drive power chairs on the market.
- **Quick-Change Shrouds** provide a quick and easy solution to swap out the color of the wheel base in a matter of seconds.
- **True Center Wheel Drive** makes the chair more intuitive to control by positioning the user directly over the drive wheels.
- **Constant Contact Casters** maintain the chair's stability going up and down inclines such as ramps.
- **Overland Suspension System** enables you to climb obstacles up to 2.5" and offers the most comfortable ride in the industry by providing independent suspension for all four casters.
- **Optional Extended Range Package** provides the freedom to go farther with increased battery capacity up to 30% vs. the standard battery option.
- **Optional Readi-Align Front Casters** ensure front caster alignment to navigate obstacles with ease.
- **Two models available**
Full size and Mini!
- **Armrest height adjustment**
quick and easy.



Readi-Align Caster



Quick-Change Shrouds

Aspire Accessories:



Cane/Crutch Holder
Walker Holder
O₂ Holder
Lap Belt (not shown)
Front Mount Basket



*Accessibility, freedom, style—
the Guardian Aspire.*

The Aspire has been thoughtfully designed to ensure a smooth, comfortable and easy ride. Our state-of-the-art Overland Suspension™ provides unsurpassed obstacle climbing. The drive wheels are center-lined to ensure driving is as intuitive as walking.

Specifications

	Microlite Ruby 11	Microlite Ruby	Aspire Full Size	Aspire Mini
Part number:	GMWRR11HB18	GMWRRCF017	Aspire F10/F11	Aspire M10/M11
Curb climbing	0.5	0.5"	3"	2.5"
Speed	3 mph	3 mph	6 mph	5.5 mph
Turning radius	19"	19"	22.5"	20"
Ground clearance	1.5"	1.5"	N/A	N/A
Drive location	rear wheel drive	rear wheel drive	mid wheel drive	mid wheel drive
Max. range	9 miles	9 miles	up to 20 miles	up to 25 miles
Max. incline	12 degrees	12 degrees	15 degrees	15 degrees
Chair length (w/footplate)	35"	35"	39.5"	38.5"
Chair length (w/o footplate)	27.5"	27"	33"	32"
Chair width	25"	19"	24.5"	22.5"
Overall weight	104 lbs/82 lbs (w/o battery)	90 lbs/69 lbs (w/o battery)	165 lbs (w/o battery)	151.75 lbs (w/o battery)
Seat weight*	32 lbs/24 lbs (w/o armrest)	19 lbs/24 lbs (w/o armrest)	30.45 lbs (w/o armrest)	30.45 lbs (w/o armrest)
Base weight	72 lbs/50 lbs (w/o battery)	66 lbs/45 lbs (w/o battery)	123.25 lbs (w/footplate)	110 lbs (w/footplate)
Weight capacity	250 lb	225 lb	300 lbs	300 lbs
Seat-to-floor height	19" - 20"	19" - 21"	21.25" - 25.25"	19.25" - 23.25"
Seat angle	0.5"	0.5"	2"	2"
Seat height adjustments	2" (3 positions)	2" (3 positions)	4" (5 positions)	4" (5 positions)
Seat Style	Mini high back	Low back	High back	High back
Upholstery	Two tone grey vinyl	Two tone grey vinyl	Two tone grey vinyl or cloth	Two tone grey vinyl or cloth
Seat Width	18"	18"	18" or 20"	18" or 20"
Seat Depth	17.5"	18"	18" or 19"	18" or 19"
Back height	14"/17.5" w/headrest	13.5"	26.5"	26.5"
Recline angle	120°	N/A	0° - 120°	0° - 120°
Armrest length	10"	10"	14"	14"
Armrest height	8"	7"	8.75" - 11.25"	8.75" - 11.25"
Armrest width adjustment	2" each side	2" each side	3" each side	3" each side
Armrest style	Flip-up	Flip-down	Flip-up, height-adjustable	Flip-up, height-adjustable
Drive tires size	8"	8"	14"	10"
Drive tires width	2"	2"	3"	3"
Drive tires type	Solid	Flat free	Flat free	Flat free
Drive wheels rim style	Alloy rim	Alloy rim	Alloy rim	Alloy rim
Rear caster size	N/A	N/A	5"	5"
Rear caster width	N/A	N/A	2"	2"
Rear caster type	N/A	N/A	Solid	Solid
Front caster size	6"	6"	6"	6"
Front caster width	1.5"	1.5"	2"	2"
Front caster type	Solid	Solid	Solid	Solid
Footrest depth	12.75"	12.75"	9"	9"
Footrest width	13.25"	13.25"	12"	12"
Footrest length adjustment	N/A	N/A	3"	3"
Footrest height	4.75"	4.75"	4" to 9"	4" to 9"
Flip-up	Yes	Yes	Yes	Yes
Frame type	Tubular	Tubular	Shrouded	Shrouded
Frame color	Red	Red	Red,blue,black,champagne	Red,blue,black,champagne
Battery type	12A pack	12A pack	Group 22 or group 24	U1 or 40 AH
Charger location	Off-board	Off-board	On-board	On-board
Charger output	1.5 A	1.5 A	8 A	4 A
Controller type	VSI from P&G	VSI from P&G	VSI from P&G	VSI from P&G
Controller output	50 A	50 A	50/70 A	50/70 A
Reimbursement code ¹	N/A	N/A	K0010/K0011	K0010/K0011

1. Coded by SADMERC

*Weight varies based on size of seat.

All specifications subject to change without notice.



www.sunrisemedical.com

Sunrise Medical
 7477 East Dry Creek Parkway
 Longmont, CO 80503
 303.218.4600
 800.333.4000
 800.263.3390 in Canada

©2004 Sunrise Medical Inc.
 479624 Rev. C