### SAVE THESE INSTRUCTIONS

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

#### ⚠ WARNING

- **DO NOT** install or use this equipment without first reading and understanding this instruction sheet. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel, before attempting to install this equipment - otherwise, injury or damage may occur.
- All four leg tips **MUST** be in contact with shower/tub floor at all times.
- **ALWAYS** inspect the shower chair to ensure that it is properly positioned and stable before using. **DO NOT** use the shower chair if it is wobbly or unstable.
- The shower chair legs have suction feet. Check the suction feet for rips, tears, cracks or wear. If any of these conditions exist, replace them **IMMEDIATELY**.
- This product should only be used with tub floors wider than 16-inches.
- Users with limited physical capabilities should be supervised or assisted when using the shower chair.
- The shower chair is **NOT** to be used as a transfer bench, transfer device or climbing device.
- These shower chairs (models 9780 and 9781) have a weight limitation of 400 lb (182 kg) each.
- Exercise caution when assembling the shower chair to avoid pinching.
- For shower chair model 9781 - Ensure that the compression buttons on the backrest are fully visible through the notch on the back of the seat.

#### ⚠ ACCESSORIES WARNING

- Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

#### ⚠ INSTALLATION WARNING

- After **ANY** adjustments, repair or service and **BEFORE** use, make sure that all parts are properly installed and secure.

#### CAUTION

- The shower chair legs have suction feet. To remove suction, carefully pull up on the lip of the suction foot to release it from the tub. If the suction feet are damaged, they **MUST** be replaced **IMMEDIATELY**.

## Assembling the Shower Chair

NOTE: For this procedure, refer to FIGURE 1.

### Attaching the Legs

1. Place the seat upside down on a flat, stable surface (floor or table).
2. Position one leg over one socket in the seat and align.
3. Push the leg down into the socket until it is fully seated and the locking tabs are through the socket tab windows. Refer to DETAIL "A".
4. Pull up on the leg to ensure that it has "locked" into place.
5. Repeat STEPS 1-4 for the remaining three legs.

### Adjusting Leg Extension Height

#### ⚠ WARNING

Ensure that the shower chair is level and stable before use.

#### To increase leg height:


1. Rotate the leg extension tip counterclockwise.
2. Use the number reference in the leg extension slot to adjust the leg height. Refer to DETAIL "B".
3. Repeat STEPS 1-2 for the remaining three legs.

NOTE: Make sure that all four legs are adjusted to the same height.


#### To decrease leg height:

1. Rotate the leg extension tip clockwise.
2. Use the number reference in the leg extension slot to adjust the leg height. Refer to DETAIL "B".
3. Repeat STEPS 1-2 for the remaining three legs.

NOTE: Make sure that all four legs are adjusted to the same height.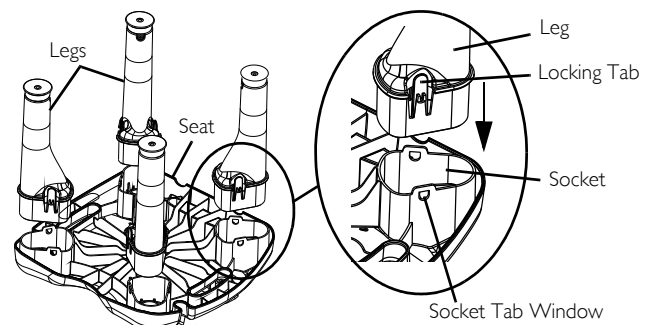

DETAIL "A" - LEGS ATTACHED TO SEAT

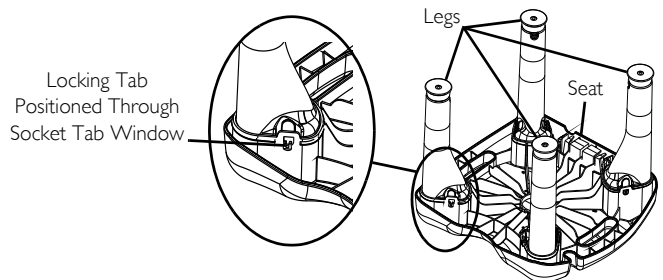DETAIL "B" - ADJUSTING LEG EXTENSION HEIGHT

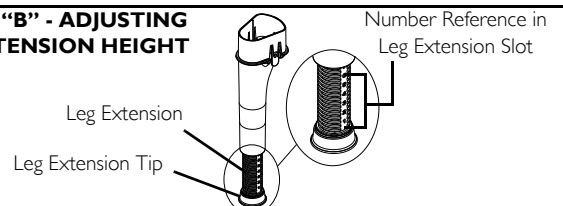
FIGURE 1 Assembling the Shower Chair

For more information regarding Invacare products, parts, and services, please visit [www.invacare.com](http://www.invacare.com)

# Installing/Removing the Backrest - Model 9781 Only

## Installing

NOTE: For this procedure, refer to FIGURE 2.

1. Before installing the backrest, attach and adjust the legs so the shower chair is level and stable. Refer to Attaching the Legs and Adjusting Leg Extension Height on page 1.
2. Position the backrest into the slot in the seat.
3. Push down on the backrest until it snaps securely into the slot in the seat.
4. Turn the shower chair upside down and look under the seat to ensure both compression buttons are fully visible through the notches on the back of the seat. Refer to DETAIL "A" of FIGURE 2.

NOTE: If both compression buttons are visible, the backrest is attached properly. If the compression buttons are not visible, apply pressure to the backrest until it snaps into the slot in the seat.

## Removing

1. Locate and depress both compression buttons with one hand.
2. Using the other hand, use the backrest handle to pull the backrest up and disengage it out of the slot in the seat.

## Cleaning

NOTE: Before using any cleaning products, test an area on the underside of the seat to make sure the product does not cause discoloration or staining.

Use water and a mild, non-abrasive cleaner to clean the shower chair.

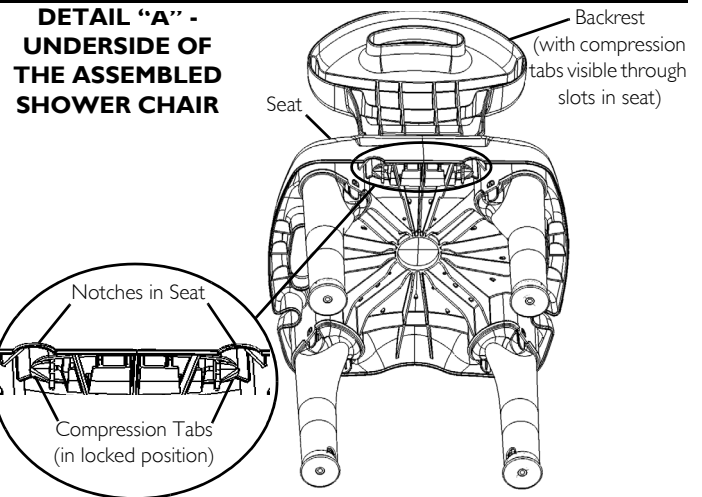
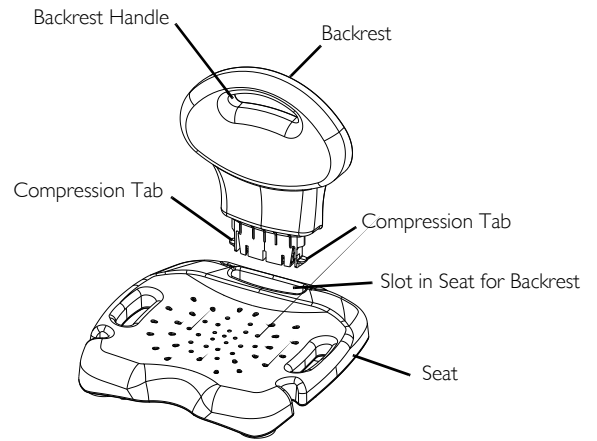
FIGURE 2 Installing/Removing the Backrest

## Limited Warranty

Invacare warrants this product to be free from defects in materials and workmanship.

**PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.**

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants this product to be free from defects in materials and workmanship for three years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will use a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

**LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS: PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.**

**THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

**THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.**

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All rights reserved. Trademarks are identified by the symbols ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
© 2009 Invacare Corporation