


INVACARE®

microAIR® MA50

SERIES THERAPEUTIC SUPPORT SURFACES

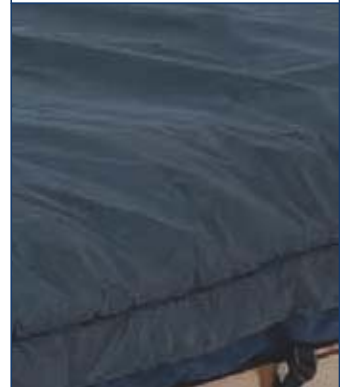

The **Invacare® microAIR® MA50 Therapeutic Support Surface** overlay is a full-featured alternating pressure overlay with four alternating pressure cycles to provide relief and comfort.

The **Invacare® microAIR® MA51 Therapeutic Support Surface** model is an alternating pressure mattress replacement system which helps reduce interface pressure to aid in the prevention and treatment of Stage I to IV pressure ulcers.

In addition to all of the features of the microAIR MA51 mattress, the **Invacare® microAIR® MA55 Therapeutic Support Surface** model improves air circulation and cools the body temperature with a unique system of low air loss released into the mattress.

Features

State of the art compressor – With solenoid valve system allows for easy set up and effective therapy cycles
Four therapy times – Can be adjusted for patient comfort to provide pressure relief

Constant pressure monitoring – Constantly monitors the air pressure in the mattress to maintain the appropriate pressure to prevent capillary occlusion (generally below 32mm/Hg)

Manual Fowler – When activated manually, 25% more air is automatically pushed into the mattress to prevent patient from bottoming out when bed is articulated

Audio and visual alarms – Warns the patient and caregiver during power outage or low pressure



Lock out button – prevents against tampering or accidental adjustments

Quick connect coupler – allows for quick and easy set up

Power outage safety – Mattress remains inflated during power outage as long as air hose is connected to the compressor

Pressure adjustment – Pressure can be adjusted by 25%, 50% or 75% to provide comfort for frail patients

HCPSC Code: E0372(MA50), E0277(MA51/MA55)

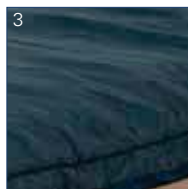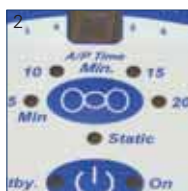

1. Easy to replace cells in case of damage
2. Choice of four therapy cycles
3. Dual layer quilted cover, denier nylon bottom



Invacare Corporation
www.invacare.com

USA

One Invacare Way
Elyria, Ohio
44035-4190
(800) 333-6900

Canada

570 Matheson Blvd. E.,
Unit 8
Mississauga, Ontario
L4Z 4G4 Canada
(800) 668-5324

©2012

Invacare Corporation.
All rights reserved.
Trademarks are identified by the symbols ™ and ®. All trademarks are owned by or licensed to Invacare Corporation unless otherwise noted. Specifications are subject to change without notification.
Form No. 10-125
Rev. 03/12
111232



Model #	Dimensions	Weight Capacity	Compressor	Alternating Time Cycle	Alarms	Limited Warranty
MA50	80" L 36" W 6" H	350 lb.	10 LPM	Adjustable	Audio and Visual	1 Year (Cover/Mattress) 2 Years (Power Unit)
MA51	80" L 36" W 8" H	350 lb.	10 LPM	Adjustable	Audio and Visual	1 Year (Cover/Mattress) 2 Years (Power Unit)
MA55	80" L 36" W 8" H	350 lb.	10 LPM	Adjustable	Audio and Visual	1 Year (Cover/Mattress) 2 Years (Power Unit)

All references to HCPCS codes: Providers are responsible for determining the appropriate billing codes when submitting claims to the Medicare program, and should consult an attorney or other advisor to discuss specific situations in further detail.

INVACARE®

microAIR® MA65

SERIES THERAPEUTIC SUPPORT SURFACE



The **Invacare® microAIR® MA65 Therapeutic Support Surface** model improves air circulation and cools the body temperature with a unique system of air released into the mattress. The MA65 mattress is an alternating pressure with on demand low air loss mattress replacement system that aids in the prevention and treatment of stage I to IV pressure ulcers.

Features

State of the art compressor – With solenoid valve system allows for easy set up and effective therapy cycles
Four therapy times – Can be adjusted for patient comfort to provide pressure relief

Constant pressure monitoring– Monitors the air pressure in the mattress to maintain the appropriate pressure to prevent capillary occlusion (generally below 32mm/Hg)

Wireless Auto Fowler – When activated, 25% more air is automatically pushed into the mattress to prevent patient from bottoming out when bed is articulated

Audio and visual alarms – Warns the patient and caregiver during power outage or low pressure

Lock out button – Prevents against tampering or accidental adjustments

Power outage safety – Mattress remains inflated during power outage as long as air hose is connected to the compressor

Cover – Anti-microbial, fluid resistant, vapor permeable cover guards against friction and shear

Pressure adjustment – Pressure can be adjusted by 50% to provide comfort for frail patients

HCPCS Code: E0277



1. Raised side rails (optional on MA65)
2. Comfort levels based on patient weight with digital display
3. Wireless auto fowler helps to prevent from bottoming out



Invacare Corporation
www.invacare.com

USA

One Invacare Way
Elyria, Ohio
44035-4190
(800) 333-6900

Canada

570 Matheson Blvd. E.,
Unit 8
Mississauga, Ontario
L4Z 4G4 Canada
(800) 668-5324

©2012

Invacare Corporation.
All rights reserved.
Trademarks are identified by the symbols ™ and ®. All trademarks are owned by or licensed to Invacare Corporation unless otherwise noted. Specifications are subject to change without notification.
Form No. 10-128
Rev. 03/12
111233



Model #	Dimensions	Weight Capacity	Compressor	Alternating Time Cycle	Alarms	Regulation Compliance	Limited Warranty
MA65, MA65RSR Raised side rail option	80" L 36" W 10" H	350 lb.	50 LPM	Adjustable	Audio and Visual	16 CFR 1632 16 CFR 1633	1 Year (Cover/Mattress) 2 Years (Power Unit)
MA65B42/ MA65B48	80" L 42"/48" W 10" H	1,000 lb.	50 LPM	Adjustable	Audio and Visual	16 CFR 1632 16 CFR 1633	1 Year (Cover/Mattress) 2 Years (Power Unit)