### SAVE THESE INSTRUCTIONS

Yes, you can.

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

### ⚠ WARNING

- **DO NOT** install or use this equipment without first reading and understanding this instruction sheet. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel, before attempting to install this equipment - otherwise, injury or damage may occur.
- The transfer bench backrest and armrest **MUST** be installed before using the transfer bench.
- **DO NOT** submerge seat in water otherwise damage to seat may result.
- Check rubber tips for rips, tears, cracks or wear. If any of these conditions exist, replace rubber tips **IMMEDIATELY**.
- Users with limited physical capabilities should be supervised or assisted when using the transfer bench.
- Use seat arm only to assist in transfers. **DO NOT** attempt to use seat arm to assist in standing in tub, otherwise tipping of the transfer bench could result.
- This product should only be used with tub floors wider than 18-5/8 inches.
- **ALWAYS** observe the weight limit on the labeling of this product. Check that all labels are present and legible. Replace if necessary.
- Most tub floors are higher than the bathroom floor. Before using, ensure that the four legs of the transfer bench inside the tub are adjusted to the same height and the two legs outside the tub are adjusted to the same height, so as to achieve a level seat. If the transfer bench has suction feet, apply all suction feet to the inside tub surface. **DO NOT** use the transfer bench if wobbly, unstable, or not level.
- After any adjustments, repair or service and before use, make sure all attaching hardware is securely tightened - otherwise injury or damage may result.
- The seat arm is **NOT** intended to support the total body weight of the user.
- **Model 9670 only** - Make sure that the compression buttons on the back support tabs are completely visible through the notches on the back of the seat before use.

### CAUTION

- If the transfer bench has suction feet, pull the tabs up to release the feet from the tub. Otherwise, permanent damage may occur.

### Set-Up

NOTE: These instructions are valid for all types of transfer benches, although models vary.

#### Attaching the Back

Perform one of the following depending on bench type:

Model 9670 (DETAIL "A"):

1. Push the tabs on the bottom of the back into the holes in the seat until the compression buttons are completely visible through the notches on the back of the seat.

Other transfer bench models (DETAIL "B"):

1. Position the back support on the back support tubes and secure with four mounting screws.
2. Align seat support tubes with the four channel guides on the underside of the frame and slide them through the channel guides until the snap buttons fully protrude through the channel guide holes.

NOTE: An audible "click" will be heard.

#### Attaching the Seat Pads/Seat

Perform the following depending on bench type:

Model 9670 (DETAIL "A"):

1. Position the seat and leg frames together upside down on a level surface (snap buttons facing inward).
2. Attach the leg frames to the seat with mounting screws.

Other transfer bench models (DETAIL "B"):

1. Position the seat pads and leg frames together upside down on a level surface.
2. Secure the end rail to the leg frame with the short mounting screws and locknuts.
3. Repeat STEPS 1-2 to attach the other end rail on the opposite end of the leg frames.
4. Align the mounting holes in the seat pads with the mounting holes in the leg frames and attach with mounting screws.

#### Attaching the Seat Arm

Perform the following depending on bench type:

Model 9670 (DETAIL "A"):

1. On the end of the transfer bench with the soap dish, remove and retain the hardware that secures the seat to the leg frame.
2. Position the seat arm against the leg frame and into the slots in the seat. Thread the mounting screws through the seat arm, the leg frame and the seat support. Secure all in place with two locknuts.


Other transfer bench models (DETAIL "B"):

1. Remove and retain the hardware that secures the end rail to the leg frame.
  2. Position the end rail against the leg frame. Position the seat arm on the outside of the leg frame.
  3. Secure the end rail and seat arm to the leg frame with two mounting screws and locknuts.
  4. Model 98071 only:
    - To install the seat arm, push the snap buttons on the seat arm and insert into the brackets. Make sure that the snap buttons fully protrude through the holes in the brackets.
- NOTE: An audible "click" will be heard.
- To remove the seat arm, push the snap buttons in and pull up.

#### DETAIL "A" - TRANSFER BENCH MODEL 9670

#### DETAIL "B" - OTHER TRANSFER BENCH MODELS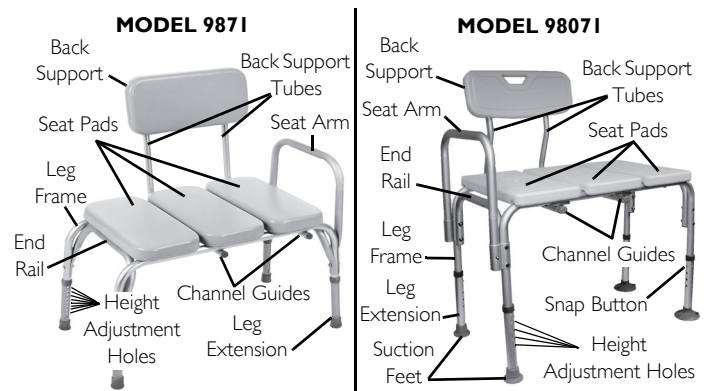

#### Adjusting Transfer Bench Height

### ⚠ WARNING

Make sure that all snap buttons protrude fully through the height adjustment holes in the leg extensions before use, otherwise injury or damage may occur.

1. Press the snap button on the leg frame.
2. Slide the leg extension up or down to the desired height until snap button protrudes fully through the height adjustment hole in the leg extension.
3. Repeat STEPS 1 and 2 for the other legs as required.

NOTE: An audible "click" will be heard.

For more information regarding Invacare products, parts, and services, please visit [www.invacare.com](http://www.invacare.com)

---

# LIMITED WARRANTY

**PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.**

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for three (3) years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts **MUST** be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. **C.O.D.** shipments will be refused; please prepay shipping charges.

**LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS, PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER). SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.**

**THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER. THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.**

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© 2008 Invacare Corporation